



Quick Des Wipes Quick Des Wipes Lemon

VAH listed*



Wet wipes for rapid disinfection

- free from aldehydes
- ready for use
- saturation liquid VAH-listed
- scent-free or with lemon scent

Product properties

Wipes soaked with alcohol containing rapid disinfectant. For the disinfection of medical devices and surfaces of all kinds. Free from aldehydes. Especially suited for all areas in which quick action is necessary.

Composition

100 g solution contain: 36,6 g Ethanol, 0,06 g Didecyldimethylammoniumchlorid, perfumes

Chemical physical data

pH-value: 7,0 +/- 1,0
Flash point: 25°C

Range of application

Acc. to Medical Devices Regulation:

Rapid disinfection of noninvasive alcohol-resistant medical devices.

Acc. to Biocidal Products Regulation:

Rapid disinfection and cleaning of alcohol-resistant medical inventory and all kinds of surfaces.

Application

Wipe alcohol-resistant surfaces until they are completely wetted and keep moist for the entire contact time.

Spectrum of activity and exposure times

Bactericidal incl. MRSA, yeasticidal acc. to VAH (dirty conditions):	5 Min.*
Inactivation of all enveloped viruses incl. Vaccinia, BVDV, HBV, HIV, HCV acc. to RKI/DVV:	1 Min.
Rota virus:	1 Min.
tuberculocidal:	2 Min.

* Soaking solution listed in 5 min.-column of the VAH-list

Delivery Forms

Single package	Sales unit	Ref.-No.
Scent-free		
Box with 120 towels Refill	1 x 1 Box	09930
pack with 120 towels	1 x 1 bag	09931
Lemon		
Box with 120 towels Refill	1 x 1 Box	09932
pack with 120 towels	1 x 1 bag	09933

Quick Des Wipes baua: Reg.-Nr. N-51832
Quick Des Wipes Lemon baua: Reg.-Nr. N-53037

Expert Opinions

Prof. Dr. Schubert, Frankfurt/Main:

Expert Opinion regarding surface disinfection in accordance with VAH Guidelines, from 05.12.2015.

Prof. Dr. Pitten, Gießen:

Expert Opinion regarding surface disinfection in accordance with VAH Guidelines from 14.03.2017.

Dr. Steinmann, Virologe, Bremen:

Expert Opinion regarding Vaccinia Viruses Effectiveness, from 27.09.2005.

Expert Opinion regarding BVDV effectiveness, from 23.09.2002.

Expert Opinion regarding Rota Virus Effectiveness, from 18.06.2008.

Prof. Dr. H.-P. Werner, Schwerin:

Expert Opinion regarding surface disinfection with mechanics from 19.04.2017

Classification according regulation (EC) Nr. 1272/2008:



Warning

Flammable liquid and vapour. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Do not breathe vapour. Use only outdoors or in a well-ventilated area. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection. For professional use only. Use disinfectants in a safe way. Always read label and product information before use.

Please notice

Open the refill package. Form a roll from the opened refill package. Put the complete refill package into the box. Pull out the first tissue from the middle of the refill roll and pull it through the opening of the application system. Close the box duly. Please refill the box only with SafeLine Quick Des Wipes.

AMPri

AMPri Handelsgesellschaft mbH
Benzstr. 16 / 21423 Winsen (Luhe), Germany
Tel. +49 (0)4171 - 8480-0 / www.ampri.de / info@ampri.de



bioSight GmbH
Brasselsbergstr. 3 / 34132 Kassel, Germany
Tel.: +49 (0)561 - 81678111

CE 0482