



VAH listed

# Quick DES Wipes N

## Alcohol-based wipes for quick disinfection

- Universal, comfortable and time-saving application
- Comprehensive, rapid effectiveness
- Particularly suitable for areas near patients



### Type of preparation

Quick DES Wipes N are alcohol-based, ready-to-use wipes for disinfection and cleaning of medical devices and medical equipment as well as surfaces in areas near patients. Quick DES Wipes N offer a universal and time-saving application when disinfecting and cleaning large areas. The wipes can be removed individually from the resealable wipe dispenser.

### Composition

100 g solution contains: 45 g ethanol

### Chemical-physical data

pH value: 5,5 - 6,5  
Flashpoint: 24°C

### Application areas

**According to the Biocidal Products Regulation (BPR):**  
Rapid disinfection and cleaning of alcohol-sensitive medical equipment and surfaces of all kinds, mainly in near patient areas.

**According to EU-Medical Devices Regulation:**  
Rapid disinfection and cleaning of non-invasive medical devices, especially for sensitive areas of medical devices. Users/patients are obligated to report any serious incident that has occurred in relation to the product to the manufacturer and the responsible authority of the EU member state in which the user/patient is established.

### Application

Wipe surfaces with Quick Des Wipes N until completely wetted and keep them moist for the required contact time. Quick DES Wipes N are intended for use with refillable wipe dispenser systems. Information on filling and processing the cloth dispenser must be taken into account. After each cloth removal the dispenser has to be closed immediately.

### Application note

Shelf life after opening the pack: 28 days

### Application times

Recommended use for surface disinfection;

bactericidal, levorocide:	2 mins
tuberculocide:	30 s
mycobactericide:	1 mins
limited virucidal:	30 s
limited virucidal PLUS:	3 mins

### Delivery forms

Single pack	Sales unit	Ref.-No.
Box of 90 wipes	1 x 1 can	09930-N
Refill pack of 90 wipes	1 x 1 bag	09931-N

**Quick Des Wipes N:** baua: Reg.-Nr. N-97773

### Expertise

Our products have been fully tested. Upon request we will be pleased to provide you with our report folder.

### Classification acc. to Regulation (EC) Nr. 1272/2008



#### Precautionary and hazard statements

Flammable liquid and vapour. If medical advice is needed, have product container or label at hand. Keep out of reach of children.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Do not breathe vapour/spray.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

For professional use only by personnel with corresponding specialist knowledge according to national directives. Use disinfectants safely.

**Always read label and product information before use.**

**AMPri**

AMPri Handelsgesellschaft mbH

Benzstr. 16 • 21423 Winsen (Luhe) • Germany

Tel. +49 (0)4171- 8480-0 • www.ampri.de • info@ampri.de



bioSight GmbH

Brasselsbergstr. 3 • 34132 Kassel • Tel.: +49 (0)561 81678111