

AMPri Handelsgesellschaft mbH

Benzstraße 16

21423 Winsen (Luhe) - Germany

Telefon: +49 (0)4171 / 8480-0 Homepage: www.ampri.de e-mail: info@ampri.de

Technical Data Sheet

Article-No.: **05562-Y**

SIZES

width

Description: MED-COMFORT

L/XL

138 cm

non woven protective coat

with PE-coating



length	138 cm						
PRODUCT DES	CRIPTION						
material	Polypro- pylene (PP)	Polyethy- lene (PE)	☐ SMS				
colour	□ blue [B]	green [G]	orange [O]	☐ red [R]	☐ white [W]	yellow [Y]	☐ light-blue [LB]
	☐ dark-blue [DB]	☐ dark-green [DG]					
closure	✓ to tie	☐ zipper	push bottons	□ velcro			
characteristics	✓ tricot-cuff	elastic wrist- cuff	✓ latex-free	hood	stand-up collar	✓ fully laminated	sewed PE- oversleeves
	shortenable by perforation	□ blocked	☐ dust-cover	☐ embossed	□ short sleeves	☐ with cardboard	single folded
REGULATORY	AFFAIRS					•	
PPE-Regulation (EU) 2016/425	category I	☐ category II	✓ category III	no PPE- article			
MD-Regulation (EU) 2017/745	☐ class I	☐ class II	☐ class III	sterile	☐ measuring fundtion	no medical device	CE
Food Contact (EG) 1935/2004	☑ acidic foods	aqueous foods	✓ fatty foods	alcoholic foods	dry foods	not approved for food- contact	7.7
CTANDARDC							
STANDARDS tested in accordance to	☑ DIN EN 14126	☑ ISO 16603	☑ ISO 16604	☑ ISO 22610	☑ ISO 22611	☑ ISO 22612	
	☑ EN ISO 13688	☑ EN 13034					

QMFORM_67.001 1/3 Herausgabe: 15.09.2021



AMPri Handelsgesellschaft mbH

Benzstraße 16

21423 Winsen (Luhe) - Germany

Telefon: +49 (0)4171 / 8480-0 Homepage: www.ampri.de e-mail: info@ampri.de


Technical Data Sheet

Article-No.: 05562-Y

Description: MED-COMFORT

non woven protective coat

with PE-coating

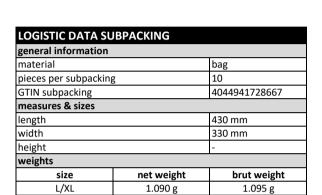

LOGISTIC DATA PALETTE				
general information				
kind of palette euro-palette				
measures & sizes				
cartons per layer	6			
layer per palette	3			
palette height	156			
weights				
size	net weight	brut weight		
L/XL	206 g	231 g		



LOGISTIC DATA OUTER PACKING				
general information				
material	carton			
subpackings per outer	10			
GTIN outer packing	4044941728674			
measures & sizes				
length	600 mm			
width	265 mm			
height	470 mm			
weights				
size	net weight	brut weight		
L/XL	10.950 g	11.450 g		



AMPri Handelsgesellschaft mbH

Benzstraße 16

21423 Winsen (Luhe) - Germany

Telefon: +49 (0)4171 / 8480-0 Homepage: www.ampri.de e-mail: info@ampri.de

Technical Data Sheet

Article-No.: 05562-Y

Description: MED-COMFORT

non woven protective coat

with PE-coating



WARNINGS AND SAFETY INFORMATION

storage / expiry	Store product in original packaging in a cool and dry place without additional weight, protect from direct sunlight. Do
date	not store near ozone sources (laser printers, copiers). The actual expiry time in use cannot be specified in general terms
	as it depends on the general conditions of use. An individual risk assessment must be carried out in each case. The
	expiry date - valid for proper storage - is stated on the packaging.

Only ever use the product for its intended use. A check/risk assessment must be carried out to ensure that the product is suitable for the intended use. Ensure that the size matches that of the user. Do not make any changes to the product. Make sure that the product has no defects and is in good condition (no holes, unstitched parts, etc.). This product is intended for single use only

disposal	The used product must be disposed of in accordance with the disposal regulations of the local waste disposal company.		
	Unused products can be disposed of with household waste.		

warnings	Choose products that are compatible with the work area.
	- The disposable item must be replaced after each use.
	- In the event of tears, punctures, etc., leave the work area and wear a new protective gown.
	- Prolonged wearing of protective gowns can lead to heat stress. Heat stress and discomfort can be reduced or
	eliminated by using suitable underwear or suitable ventilation equipment.
	- These garments are flammable - keep away from fire.
	- In the event of damage to the product, leave the workplace immediately.
	- The user must not remove the protective gown while still in the danger zone.

donning and doffing	WAY

instructions

- WAY OF PUTTING ON:
- Make sure that the size matches that of the user. Do not make any changes to the product.
- Make sure that the product has no defects and is in good condition (no holes, unstitched parts, etc.).
- Open the gown and put it on. Make sure that the garment does not tear. Close the gown by wrapping the waistband around your body

body and fastening it with a bow.

REMOVAL METHOD:

- Undo the waist and collar ties and remove the gown.
- The disposal of contaminated clothing is regulated by national or local laws.

Do not dispose of contaminated protective clothing in household waste.

- The protective properties are only effective if the article is correctly put on and closed.

3/3

- Protect uncovered parts of the body (hands, respiratory tract, feet) with protective gloves, boots, mask, etc., attach them to the gown (with adhesive strips if necessary) and ensure that they offer the same level of protection to ensure full-body protection.

rev-no.: 6

date: 22. 10.2024

changes and errors excepted

QMFORM_67.001 Herausgabe: 15.09.2021