



Soaking solution
VAH listed

Quick Des Wipes Maxi

alcohol wipes for rapid disinfection

- free from aldehydes
- ready for use
- soaking solution VAH-listed
- sheet size 30 x 32 cm

Product properties

Wipes soaked with alcohol containing quick disinfectant. For the disinfection of medical products and surfaces of all kinds. Free from aldehydes. Especially suited for all areas in which quick action is necessary.

Composition

Active ingredient in 100 g solution: 36.6 g Ethanol, 0.06 g Didecyldimethylammoniumchloride. Contains perfumes.

Chemical physical data

pH-value: 7,0 +/- 1,0
Flash point: 25°C

Range of application

Acc. to Medical Devices Directive (MDD):
Rapid disinfection of noninvasive alcohol resistant medical devices.

Acc. to Biocidal Products Regulation:
Rapid disinfection and cleaning of alcohol resistant medical inventory and all kinds of surfaces.

Application

Wipe alcohol-resistant surfaces until they are completely wetted and keep moist for the entire contact time. Close the lid of the dispenser after taking out a wipe.


Spectrum of activity and exposure times

Bactericidal incl. MRSA, yeasticidal acc. to VAH (high bioburden level):	2 Min.*
Inactivation of all enveloped viruses incl. Vaccinia, BVDV, HBV, HIV, HCV acc. to RKI/DVV:	1 Min.
Rota virus:	1 Min.
tuberculocidal:	2 Min.

* Soaking solution listed in 5 min.-column of the VAH-list

Delivery Forms

Single package	Sales unit	Ref.-No.
Set with 90 wipes and dispenser	1 x 1 Set	09928
Refill pack with 90 wipes	1 x 1 bag	09929

 bioSight GmbH
Brasselsbergstr. 3 / 34132 Kassel, Germany
Tel.: +49 (0)561 - 81678111

Expert Opinions

Prof. Dr. Schubert, Frankfurt/Main:

Expert Opinion regarding surface disinfection in accordance with DGHM/VAH Guidelines, from 28.05.2012.

Prof. Dr. Pitten, Gießen:

Expert Opinion regarding surface disinfection in accordance with DGHM/VAH Guidelines from 07.08.2012. Expert Opinion Regarding Tuberculocidal effectiveness, from 07.08.2012

Dr. Steinmann, Virologe, Bremen:

Expert Opinion regarding Vaccinia Viruses Effectiveness, from 27.09.2005.

Expert Opinion regarding BVDV effectiveness, from 23.09.2002.

Expert Opinion regarding Rota Virus Effectiveness, from 18.06.2008.

Classification according regulation (EC) Nr. 1272/2008:



Warning

Flammable solid. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

Use disinfectants in a safe way. Always read label and product information before use.

Filling of the wet cloth dispenser

Put the Safeline Quick Des Wipes Maxi refill including bag direct into the wet cloth dispenser. Open the refill. Pull out the first tissue from the middle of the refill roll and pull it through the opening in the lid of the application system and close the lid properly. Please refill the dispenser only with SafeLine Quick Des Wipes Maxi.

AMPri

AMPri Handelsgesellschaft mbH
Benzstr. 16 / 21423 Winsen (Luhe), Germany
Tel. +49 (0)4171 - 8480-0 / www.ampri.de / info@ampri.de