



Suction Clean

Additional Cleaner for Dental Suction Systems

- strong cleaner power
- low foaming



Product properties

Low foaming cleaning concentrate for dental suction systems.

Composition

Organic acids, < 5 % phosphonates.

Chemical physical data

Appearance of the concentrate: clear, yellow solution
 pH-value: approx. 1,7 +/-0,5
 Density (at 20°C): approx. 1,058 g/ml

Range of application

For the regular, additional cleaning of dental suction systems, spittoon bowls and run-off tubes.

Application

Apply SafeLine Suction Clean once or twice a week. Depending on level of soiling use 50 ml to 100 ml Descosuc Cleaner and fill up to 1 litre with water. Prepare 2 litre of fresh working solution before cleaning procedure (cleaning can be done e.g. during lunch break). Extract 2/3 of the working solution through the suction tubes and pour 1/3 of the working solution into the dental sink. The best cleaning result will be achieved at an exposure time of ca. 1.5 to 2 hrs. After exposure time rinse carefully with clear water in order to completely remove dissolved soiling from the tubes. Do not leave the working solution in the suction system over night. Further information regarding the application can be found on the detailed product information sheet. According to the EU Medical Device Regulation, users/patients are obligated to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the EU Member State in which the user/patient is established.

Concentration and exposure time

normal organic soiling: 5 % (50 g/L).
 High organic soiling: up to 10 % (100 g/L).
 Contact time: 1.5 - 2 hours, depending on soiling level

Delivery Forms

Single package	Sales unit	Ref.-No.
5 litres container	1 x 5 L	09982

Classification according regulation (EC) Nr. 1272/2008:



Danger.
 Glycolic acid (Hydroxyacetic acid), 1-Hydroxyethane-1,1-diphosphonic acid. Causes severe skin burns and eye damage. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Call a POISON CENTER/doctor. For professional use only.

Use disinfectants in a safe way. Always read label and product information before use.